medela

symphony[®] & lactina[®] breastpump kit

kit de extractor de leche
materna symphony[®] y lactina

#1 choice
of hospitals
& mothers™

EN instructions | ES instrucciones

-  for use with symphony breastpumps
-  for double or single pumping
-  para usarse con extractores de leche symphony
-  para extracción doble o simple

Not made with natural rubber latex or DEHP.

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Table of Contents

1. Important Safeguards.....	4
2. Accessories.....	5
3. Cleaning Overview	6-7
4. Wash – Before first use & after each use.....	8-10
5. Sanitize – Before first use & once per day.....	12-13
6. Tubing Care	14-15
7. Assembly of the Pump Kit.....	16-25
7A. Symphony®	16-19
7B. Lactina®.....	20-23
7C. Manual Breastpump Assembly	24-25
8. Breastshield Sizing	26
9. Storing, Preparing & Feeding Breastmilk	27
9A. Storing Breastmilk	27
9B. Preparing Breastmilk.....	27
9C. Feeding Breastmilk.....	27
10. Troubleshooting	28-29
11. Supplemental Information.....	30

IMPORTANT

- Plastic bottles and component parts become brittle when frozen and may break when dropped.
- Bottles and component parts may become damaged if mishandled, e.g. dropped, over-tightened, or knocked over.
- Take appropriate care in handling bottles and components.
- Do not use the breastmilk if bottles or components become damaged.

Symphony® Intended Use

The Symphony Breastpump is a powered breastpump to be used by lactating women to express and collect milk from their breast.

Symphony Product Description

Symphony is a multi-user, hospital grade electric breastpump that includes 2-Phase Expression® technology and is capable of single and double pumping.

Lactina® Intended Use

Mothers milk is the best nutrition for your baby. The Lactina Breastpump is a clinically proven pump suitable for frequent and long term pumping.

Lactina Product Description

The Lactina Breastpump is a powered breastpump:

- To be used by lactating women to express and collect milk from their breast.
- To initiate and maintain a milk supply as well as collect breastmilk at work or during other absences from the baby.
- To initiate and maintain a milk supply if direct breastfeeding is not possible.
- To relieve engorgement.
- For use in a hospital or home.

The warning symbol identifies all instructions that are important to safety. Failure to observe these instructions can lead to injury or damage to the breastpump. When used in conjunction with the following words, the warning symbols stand for:

WARNING Can lead to serious injury or death.

CAUTION Can lead to minor injury.

NOTE Can lead to material damage.

TIP Useful or important information that is not related to safety.

PLEASE SAVE THESE INSTRUCTIONS.

Questions? Visit www.medela.com or call us at 1-800-435-8316.

1. Important Safeguards

When using electrical devices, basic safety precautions should always be followed, including the following:

READ ALL INSTRUCTIONS BEFORE USING THIS PRODUCT.

WARNING: To avoid health risk and reduce the risk of injury:

- Breastpump kits are a personal care item intended for use by a single user only and should not be shared between users.
- Inspect the breastpump kit components prior to each use.
 - Do not use the components if they are damaged.
 - Do not use the components if dirt, mold or other contamination is visible.
- Inspect all appropriate pump components before each use.
- Clean and sanitize all parts that come in contact with your breast and breastmilk prior to first use. Sterile kits are ready to use and do not require washing prior to first use.
- Do not modify any breastpump kit components.
- Wash all parts that come in contact with your breast and breastmilk after every use.
- Do NOT continue pumping for more than 2 consecutive pumping sessions if no results are achieved.
- Do not thaw frozen breastmilk in a microwave or in a pan of boiling water.
- If tubing becomes moldy, discontinue use and replace tubing.
- If breastmilk backup occurs, you must clean tubing before your next pumping session.
- If problems or pain occur, consult your lactation consultant or physician.

CAUTION: Can lead to minor injury:

- Pump only with the breastpump in an upright position.
- Do NOT shorten tubing between breastshield and pump.
- Make sure the voltage of the device is compatible with the power source.
- Plug the power cord into the breastpump first and then into the wall socket.
- Do NOT use abrasive cleaners/detergents when cleaning breastpump or breastpump parts.
- Never put breastpump motor in water or a sterilizer, as you can cause permanent damage to the breastpump.
- Do not attempt to remove the breastshield from your breast while pumping. Turn the breastpump off and break the seal between your breast and breastshield with your finger, then remove breastshield from your breast.
- If pumping is uncomfortable or causing pain, turn the unit off, break the seal between the breast and the breastshield with your finger and remove the breastshield from your breast.
- Do not store wet or damp parts as mold may develop.
- Do not use the breastpump kit with a non-compatible breastpump.
- Do not use the breastpump kit for any other purpose than expressing breastmilk.

2. Accessories

Symphony® Membrane Cap	8100769
Symphony Protective Membrane	8200094
PVC Tubing - Symphony, Lactina®, Classic™	8007213
PersonalFit™ Connectors	87071
PersonalFit 21 mm Breastshield	87072
PersonalFit 24 mm Breastshield	87073
PersonalFit 27 mm Breastshield	87274
PersonalFit 30 mm Breastshield	87075
PersonalFit 36 mm Breastshield	87084
Replacement Membranes	87088
Replacement Valves & Membranes	87089
Lactina Piston Cylinder Assembly	8007190
Lactina Pump Connector with port plug	8007195

Authentic Medela Spare Parts are made with the quality materials and workmanship you expect and trust, only from Medela.

Non-Medela spare parts can vary significantly in terms of product design, materials, and workmanship; all of which may affect the performance of your Medela breastpump.

Quality & performance
you can trust

3. Cleaning Overview

Note

It's important for you to do the following before using Symphony® or Lactina® for the first time*:

1. For Symphony models that include a rechargeable battery, fully charge the battery 12 hours uninterrupted.
2. Separate all parts.
3. Clean - see instructions:
 - Wash – section 4, pages 8-10
 - Sanitize – section 5, pages 12-13

Note

*Sterile kits are ready to use and do not require washing prior to first use.

Breastshield assemblies come assembled. Separate all parts that will come into contact with the breast and breastmilk before cleaning.

Parts to clean:

- Breastshields
- Breastmilk bottle/collection containers
- Lids
- Connectors
- Membranes
- Valves
- Piston
- Cylinder

Kit components may vary from pictures in this instruction manual.

	Breastpump kit & bottles/containers 	Tubing 	Lactina® piston/cylinder 
Before 1st use*	✓ wash ✓ sanitize		
After each use	✓ wash		
Once per day	✓ sanitize		
As needed		✓ wash only if milk or condensation is in tubing	✓ wash every few pumping sessions

Detailed cleaning instructions on pages 8-15:

- Wash – pages 8-10
- Sanitize – pages 12-13
- Tubing care – pages 14-15
- Lactina piston cylinder cleaning care – page 10

Note

*Sterile kits are ready to use and do not require washing prior to first use.

Warning

Breastpump kits are a personal care item intended for use by a single user only and should not be shared between users.

4. Wash – Before first use and after each use*

Supplies needed:

- Mild dish soap
- Clean dish towel or soft brush
- Clean sink or bowl
- Drinking-quality water

Parts to wash:

- Breastshields
- Breastmilk bottles/containers
- Lids
- Connectors
- Membranes
- Valves
- Piston
- Cylinder

Note

*Sterile kits are ready to use and do not require washing prior to first use.

Warning

Breastpump kits are a personal care item intended for use by a single user only and should not be shared between users.

- Separate parts for washing.


Caution

Separate and wash all parts that come in contact with breastmilk immediately after use to avoid dry up of breastmilk residue and prevent growth of bacteria.

Note

- Wash breastpump kit parts after each use.
- Take care not to damage parts of the pump kit during cleaning.
- **It is not necessary to wash tubing, membrane cap or protective membrane prior to first use.**
- **Only wash tubing if condensation or milk is present.**

Wash in dishwasher

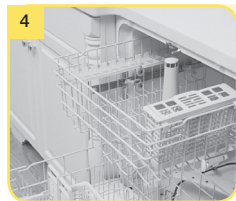- Wash all separated parts on top rack of dishwasher.
- Allow all breastpump parts to air dry in a clean area.
- Store dry parts in a clean, cool place when not in use.

Note

If the individual components of the pump kit are cleaned in a dishwasher, parts may be discolored by food pigments. This will not impact part function.

Rinse all separated parts that came in contact with breast and breastmilk in cool water in order to remove breastmilk residue.

Wash in sink


- Soak all separated parts in warm soapy water for 5 minutes.
- Wash each part with a clean dishcloth or soft brush.
- Rinse all separated parts with clear water.

Caution

Only use drinking-quality tap or bottled water for cleaning.

You may also wash your kit parts by following the instructions on the Quick Clean™ Breastmilk Removal Soap bottle. (sold separately)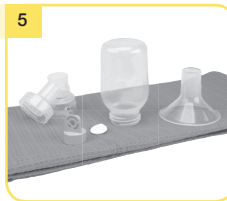

- Place parts on a clean surface and/or towel.
- Allow all parts to air dry.
- Store dry parts in a clean, cool place when not in use. Do NOT store wet or damp parts.

Note

Store the pump kit in a clean bag/container until next use.

4. Wash (cont.)

Lactina Piston/Cylinder Cleaning

Care for your piston and cylinder after every few pumping sessions with the Lactina breastpump or manual breastpump.

- Wash the cylinder, seal, and vacuum regulator ring with warm soapy water.
- Rotate the piston in the cylinder.
- If your piston/cylinder comes in contact with breastmilk remove and wash with breastpump kit.
- Do not use Quick Clean™ Micro-Steam™ bag to sanitize the piston and cylinder.

5. Sanitize – Before first use and once per day*

Supplies needed:

- Clean pot for boiling water
- Drinking-quality water
- Tongs

Parts to sanitize:

- Breastshields
- Breastmilk bottles/containers
- Lids
- Valves
- Connectors
- Membranes

Note

*Sterile kits are ready to use and do not require washing or sanitizing prior to first use.

Warning

Breastpump kits are a personal care item intended for use by a single user only and should not be shared between users.

- Separate all parts that come in contact with breastmilk.

Note

Sanitize breastpump kit parts once daily.

- Wash hands thoroughly.


- Fill a pot with enough water to cover all parts.
- Bring water to a boil.
- Place parts in boiling water for 10 minutes.

Note

If you notice a white residue on your parts after boiling, you may have a high mineral content in your water. Remove residue by wiping parts with a clean washcloth and air dry. We recommend that you boil parts with distilled water to prevent substantial mineral build-up over time which may compromise your parts.

Allow water to cool and gently remove parts from water with tongs.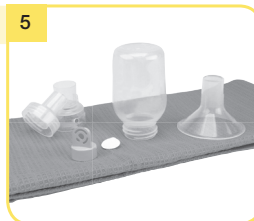

- Place parts on a clean surface and/or towel.
- Allow all parts to air dry.
- Store dry parts in a clean, cool place when not in use. Do NOT store wet or damp parts.

Note

Store the pump kit in a clean bag/container until next use.

You may also sanitize your kit parts by following the instructions on Medela's Quick Clean™ Micro-Steam™ bag, (sold separately) Not for use with piston and cylinder.

6. Tubing Care

Supplies needed:

- Mild dish soap
- Clean dish towel
- Clean sink or bowl
- Drinking quality water

Parts needed:

- Tubing
- Symphony® or Lactina® pump
- Membrane cap (Symphony)
- Protective membrane (Symphony)
- Piston (Lactina)
- Cylinder (Lactina)
- Pump connector (Lactina)

- Inspect tubing after each pumping session.

Note


Only wash tubing if condensation or milk is present.

Caution

- Do not store wet or damp parts as mold may develop.
- If tubing becomes moldy, discontinue use and replace tubing. Contact Medela Customer Service at customer.service@medela.com or call 1-800-435-8316.

1 Symphony

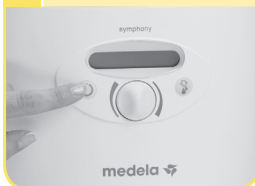
Turn off breastpump.

2 Symphony

Unplug breastpump from power source.

3 Symphony

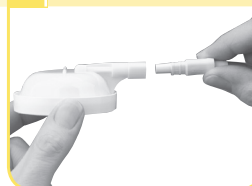
Remove tubing, protective membrane, and membrane cap. Clean following steps in section 4.

1 Lactina


Turn off breastpump.

2 Lactina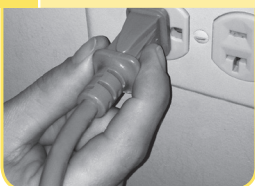

Unplug breastpump from power source.

3 Lactina

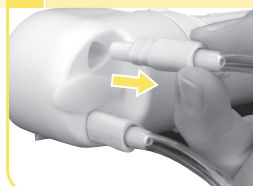Remove tubing from port on pump connector. Follow cleaning instructions for remaining kit parts in section 4.

4

Remove tubing from breastshield.

5

Rinse tubing in cool water to remove breastmilk.

6

- Wash tubing in warm soapy water.
- Rinse tubing with clear water.
- Hang to air dry or shake out any drops of water from the tubing and let the pump run with tubing attached for 1-2 minutes until dry.

Warning

Breastpump kits are a personal care item intended for use by a single user only and should not be shared between users.

Warning

Refer to breastpump instructions for use for important safeguards and operating information.

7. Assembly of Pump Kit - Symphony®

Tip

Carry out all the steps with care and assemble the pump kit correctly for optimum performance.

Parts Needed:

- Tubing
- Symphony® pump
- Breastshields
- Breastmilk bottles/containers
- Lids
- Connectors
- Valves
- Membranes
- Membrane cap
- Protective membrane

Note


- Use Authentic Medela Spare Parts only. See page 5 for details.
- Check pump kit components for wear or damage before use and replace if necessary.
- All components must be completely dry before use for proper performance and to prevent damage to the pump.

7A. Symphony

Caution


Wash hands thoroughly with soap and water before touching breastpump, kit and breasts and avoid touching the inside of containers or lids.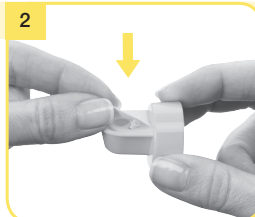

Carefully snap membrane onto yellow valve until membrane lies flat.

Push the breastshield onto the connector.

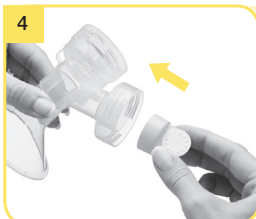Push yellow valve onto inner ring of connector bottom.


Screw connector onto the bottle/ container.

Insert the tubing into the corresponding opening in the connector.

Place the protective membrane onto a flat surface and carefully push the membrane cap down until it engages.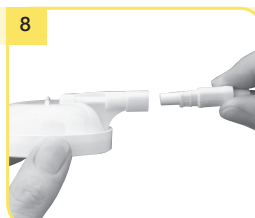

Insert the tubing adapter into the opening in the membrane cap.

Warning

Refer to pump instructions for use for important safeguards and operating information.

7. Assembly of Pump Kit - Symphony® (cont.)

Open the lid of the breast pump.
→ Press the oval button on the top of the Symphony® and lift the handle.

Position the membrane cap firmly on the breast pump membrane to form a seal. Center the tubing connector in the recess in the pump housing.
→ Make sure that the membrane cap engages and lies flush to the surface.

Note

If single pumping, only seat one membrane cap.

Center the tubing connector in the recess in the pump housing.
→ Make sure that the membrane cap engages and lies flush to the surface.

Warning

Refer to pump instructions for use for important safeguards and operating information.

Close the lid.

Note

• **For step 12:**
Always close the lid when pumping. The lid keeps the membrane cap pushed down to prevent a loss of vacuum that could occur if the membrane cap came off.

Center breastshields over your nipples and start pumping. (Caution: pushing shields too firmly onto breasts may affect milk flow.)

To turn the pump on, press the power button. You will be starting in the Stimulation Phase. After 2 minutes, the pump will automatically change to the Expression Phase.

If let-down occurs before the Expression Phase begins, press the let-down button to begin the Expression Phase.

When your pumping session is over, store your breastmilk and clean your kit. See sections 3-6 for cleaning instructions. Refer to section 9 for storage guidelines.

7. Assembly of Pump Kit - Lactina®

Tip

Carry out all the steps with care and assemble the pump kit correctly for optimum performance.

Parts Needed:

- Tubing
- Lactina® pump
- Breastshields
- Breastmilk bottles/containers
- Lids
- Connectors
- Valves
- Membranes
- Piston
- Cylinder
- Pump Connector

Note


- Use Authentic Medela Spare Parts only. See page 5 for details.
- Check pump kit components for wear or damage before use and replace if necessary.
- All components must be completely dry before use for proper performance and to prevent damage to the pump.

7B. Lactina


Caution

Wash hands thoroughly with soap and water before touching breastpump, kit and breasts, avoid touching the inside of containers or lids.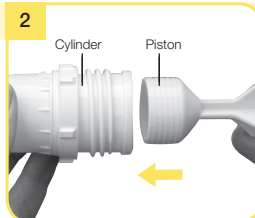


Slide the vacuum regulator ring onto cylinder. Push handle end of piston into cylinder.

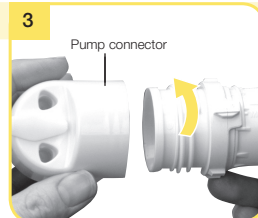
Screw cylinder tightly into Lactina pump connector.

Attach pump connector to Lactina, and rotate cylinder down into position.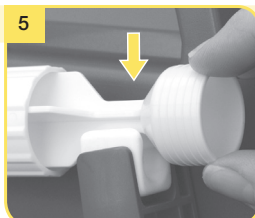


Slip piston into pumping arm cradle, and push down until snug.

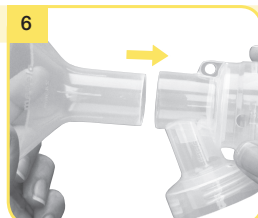Securely attach breastshields to connectors.

Snap membranes onto yellow valves until membranes lie flat.

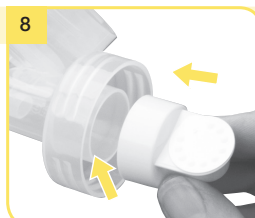Push yellow valves onto inner rings of connector bottoms.


Warning

Refer to breastpump instructions for use for important safeguards and operating information.


7. Assembly of Pump Kit - Lactina® (cont.)

Screw connector onto the bottle/container.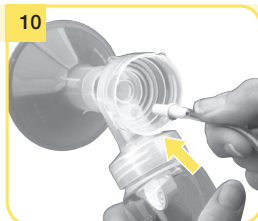


Insert pale yellow round ends of tubing into openings in back of connectors.

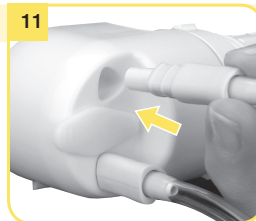Insert the triangular two-toned yellow ends of tubing into ports on pump connector.

Center breastshields over your nipples and start pumping. (Caution: pushing shields too firmly onto breasts may affect milk flow.)

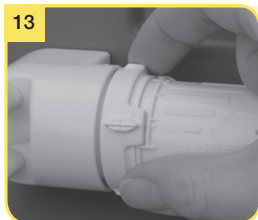Turn pump on and set vacuum regulator ring to "MIN." Rotate vacuum regulator towards max to adjust for comfort once pumping begins.

Note

- If single pumping, insert port plug into unused port.

When your pumping session is over, store your breastmilk and clean your kit. Refer to sections 3-6 for cleaning instructions. Refer to section 9 for breastmilk storage guidelines.

7. Assembly of Pump Kit - Lactina® (cont.)

Tip

Carry out all the steps with care and assemble the pump kit correctly for optimum performance.

Parts Needed:

- Breastshield
- Breastmilk bottles/container
- Lid
- Connector
- Valve
- Membrane
- Piston
- Cylinder


Note


- Use Authentic Medela Spare Parts only. See page 5 for details.
- Check pump kit components for wear or damage before use and replace if necessary.
- All components must be completely dry before use for proper performance and to prevent damage to the pump.

7C. Manual Breastpump Assembly

For manual breastpumping follow the instructions below.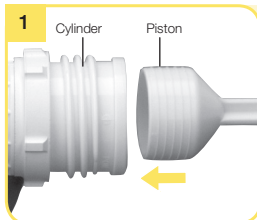


Slide the vacuum regulator ring onto the cylinder. Push handle end of piston into the cylinder.

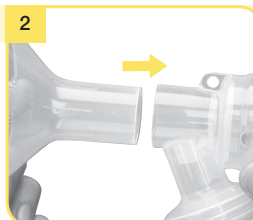
Securely attach breastshield to connector.


Snap membrane onto yellow valve until membrane lies flat.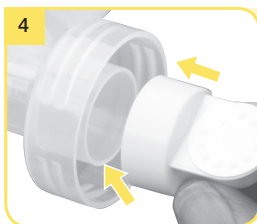

Push yellow valve onto inner ring of connector bottom.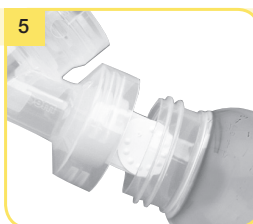


Screw connector onto the bottle/container.

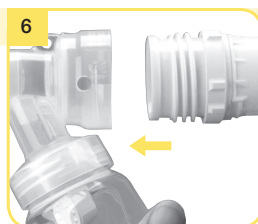Screw cylinder into back of breastshield.

Set vacuum regulator ring to "MIN".

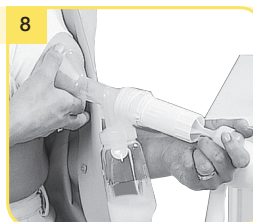Center breastshield over your nipple and start pumping by pushing and pulling on the piston end.

When your pumping session is over, store your breastmilk and refer to section 9. Refer to sections 3-6 for cleaning instructions.

8. Breastshield Sizing

Caution

- While some discomfort may be felt when first using a breastpump, using a breastpump should not cause pain.
- If you are experiencing discomfort at the base of the nipple due to rubbing of your breast tissue with the breastshield tunnel, use of a lubricant such as Tender Care™ Lanolin may be beneficial.
- For assistance with correct breastshield sizing and comfort please visit www.medelabreastshields.com or see a lactation consultant / breastfeeding specialist.

Correctly fitting breastshields

Properly fitting breastshields will ensure comfort and efficient milk expression. To determine if you have the correct size, look at the following diagram:

- The nipple should be centered in the tunnel of the shield.
- The nipple should move freely without rubbing when the pump is turned on.
- Minimal or no part of your areola should be pulled into the tunnel of the breastshield.
- You should see a gentle, rhythmic motion in the breast with each cycle of the pump.
- After pumping, your breast should feel less full with no areas of hardness.

Correct fit

PersonalFit™ Breastshields

21 mm
Item 87072

24 mm
Item 87073

27 mm
Item 87274

30 mm
Item 87075

36 mm
Item 87084

Choosing the right size PersonalFit™ breastshield:

- 1 Determine the size you're currently using. If you're not sure, look for the size embossed on your breastshield (see picture).

24 mm breastshields are provided with Medela breastpumps.

- 2 While pumping, compare your fit to the images below as a sizing guideline.

Correct fit

Your breastshield is **too small**; try a larger size

Your breastshield is **too large**; try a smaller size

PersonalFit™ Breastshields are available at many local retailers or visit www.shopmedela.com.

9. Storing, Preparing & Feeding Breastmilk

9A. Storing breastmilk

Freshly Expressed Breastmilk Storage Guidelines (For Healthy Term Babies)

Room Temperature	Cooler with Ice Pack	Refrigerator	Freezer
4–6 hours at 66–78 °F (19–26 °C)	24 hours at 59 °F (15 °C)	3–8 days at 39 °F or lower (4 °C)	6–12 months 0–4 °F (-18–20 °C)

References: www.BreastmilkGuidelines.com

9B. Preparing breastmilk

Caution

- Do not thaw frozen breastmilk in a microwave or in a pan of boiling water.
- Do not microwave breastmilk. Microwaving can cause severe burns to baby's mouth from hot spots that develop in the milk during microwaving. (Microwaving can also change the composition of breastmilk.)
- Thaw breastmilk overnight in the refrigerator. Thawed breastmilk is safe in the refrigerator for 24 hours. Do not refreeze thawed breastmilk.
- Quickly thaw breastmilk by holding the bottle under warm running water.
- Place the sealed bottle in a bowl of warm water for 20 minutes to bring it to body temperature.
- If you are adding expressed breastmilk to a container of already frozen breastmilk, make sure to add a lesser amount than the already frozen amount.

9C. Feeding breastmilk

It is recommended that breastfeeding is well established prior to bottle feeding your baby.

- Always inspect the bottle, nipple and other components immediately before and after each use. If nipple appears cracked or torn, discontinue use immediately.
- To prevent possible choking hazard, test strength of nipple by pulling on bulb portion of the nipple.
- Do not attempt to enlarge the nipple hole.
- Infants must not be bottle fed without adult supervision.
- Nipple should not be used as a pacifier.

10. Troubleshooting

If the breastpump is not turning on	<p>Check whether electrical outlet power/battery is available. Check that the device is switched on. Check whether program card is inserted correctly in the card compartment on the back of the breastpump. (Symphony® breastpump only)</p>
If there is low or no suction	<p>Assemble the pump kit in accordance with the instructions in Section 7.</p> <ul style="list-style-type: none"> • Check whether all connections of the pump kit and breastpump are secure. • Make sure that the breastshield forms a complete seal around the breast. • The white valve membrane must lie flat against the yellow valve head. • Check that the valve head and membrane are clean and not damaged. • The ends of the tubing must be firmly connected to the connector and the opening in the membrane cap. (Symphony® breastpump only) • The protective membrane and membrane cap should be correctly assembled and the membrane cap must be firmly fitted to the breastpump membrane to form a seal. (Symphony breastpump only) • The protective membrane must be intact. (Symphony® breastpump only) • Do not kink or pinch the tubing during expression. • All components must be completely dry. <p>Valve and Membrane Check</p> <ul style="list-style-type: none"> • Remove and separate valve from the membrane. • Inspect the valve for chips or cracks. Inspect the membrane for holes or tears. If flawed or damaged, replace the valve and/or membrane prior to pumping. • Wash valve and membrane per Cleaning Instructions in Sections 3-6. • Reassemble. Make sure the membrane lays flat on the valve, and if it does not, replace the valve and/or membrane prior to pumping. <p>Breastshield and Breastshield Connector Check</p> <ul style="list-style-type: none"> • Inspect breastshield and breastshield connector to make sure there are no cracks and they securely attach to each other. • Make sure the breastshield forms a complete seal around the breast. <p>Other Checks</p> <p>Inspect the following to check for damage and secure connections.</p> <ul style="list-style-type: none"> • Secure tubing adapter to the breastshield connector. • Test vacuum. If suction is not improved, contact Medela Customer Service at 1-800-435-8316. • Inspect piston/cylinder and refer to piston/cylinder cleaning in Section 4. (Lactina® Breastpump only) <p>Tubing Check</p> <ul style="list-style-type: none"> • If damaged replace prior to pumping. • Inspect tubing for damage and secure connections. • Check that the ends of the tubing are firmly connected to the connector and the opening in the membrane cap. • If single pumping, make sure only one membrane cap is seated. (Symphony® Breastpump only) • If single pumping, insert port plug into unused port. (Lactina® Breastpump only) <p>Piston/Cylinder/Tubing Check (Lactina® Breastpump Only)</p> <ul style="list-style-type: none"> • Make sure the ends of the tubing are fit securely into the back of the breastshield and into the opening on the pump connector. • Make sure the rubber seal is correctly attached onto piston. • Make sure the piston/cylinder is correctly attached onto the pump connector or breastshield. • Make sure the pump connector and piston/cylinder are assembled properly onto the Lactina. • If single pumping, make sure the port cap is inserted into the unused opening of cylinder holder. <p>Take care not to kink or pinch tubing.</p>

If there is an overflow	<ol style="list-style-type: none"> 1. Place the pump set into the bottle holder. 2. Turn off the pump and unplug the power cord from the electrical outlet. 3. Use a damp (not wet) cloth to wipe the pump and membrane area. 4. If there is milk in the tubing: Remove tubing and clean in accordance with Sections 4 & 6. If there is condensation in the tubing: Let the pump run with tubing attached for 1–2 minutes or until dry. 5. When components are completely dry, reassemble breastpump and accessories set.
Condensation or Milk in Tubing	<ul style="list-style-type: none"> • Turn off the pump and unplug from power source. • Use a damp (not wet) cloth to wipe the pump. • Refer to cleaning instructions "Tubing Care" Section 6. • If necessary, clean the pump kit. • When components are completely dry, reassemble breastpump and kit.
No "Let-Down" or Breastmilk Expression	<ul style="list-style-type: none"> • Ensure your breastpump kit is assembled correctly and suction is available. • Relax and take a 10-15 minute break if let-down is not achieved. • Consult with a healthcare or lactation professional if expression does not occur after 2 consecutive pumping sessions.

If you have not resolved the problem with your breastpump or you have further questions, please contact Medela Customer Service at 1-800-435-8316 or fill out the contact us form at www.medelabreastfeedingus.com.

11. Supplemental Information

Supplemental Information

Following are some common breastfeeding related conditions. If you experience any of these symptoms, contact a healthcare professional or lactation professional.

	Symptom	Potential Causes
Engorgement	Breasts are hard and uncomfortable, possibly with reddened areas.	Milk not draining properly from breast. Milk can drain back into the tissue causing areas to swell and become tender.
Blocked/ Plugged Ducts	Area of breast will look reddened and may be tender to touch.	Milk not being drained from a specific duct. The area becomes "clogged" up and milk is then prevented from flowing.
Mastitis	It often follows engorgement but can occur suddenly. The initial symptoms are similar to flu – fatigue, headache, muscle aches, fever and localized breast tenderness. It can be in one or even both breasts and needs immediate treatment.	A bacterial infection in the breast tissue which is often accompanied by cracked nipples.
A Healthcare Professional or Lactation Professional can give you guidance in breastfeeding your baby and using a breastpump. Their guidance is valuable in terms of the lifetime of health benefits for both you and your baby. A listing of lactation professionals are available at www.medelaLocator.com or calling 1-800-TELL-YOU (1-800-835-5968).		

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1908210 C 0315 © 2015 Medela, Inc. Printed in the U.S.A./Impreso en los EE. UU.
Medela, Lactina and Symphony are registered trademarks of Medela Holding AG.
Micro-Steam are trademarks of Medela, Inc.
Medela, Inc.
Medela, Lactina y Symphony son marcas comerciales registradas de Medela Holding AG.
Micro-Steam son marcas comerciales de Medela, Inc.