IMPORTANT! Ensure that your Lansinoh® Breastmilk Storage Bag has **white and purple flowers** and a **white line above the logo** as shown in the image. If your bag does not have this design, it is not compatible with pumping directly into the bag.

NOTE: Pumping directly into bags requires two hands to safely remove milk storage bags. For best results use with a hands free pumping bra such as the Lansinoh® Simple Wishes® Bra.

Lansinoh.

Pump Into Bag Options for Time-Saving Convenience

Directions for use with Lansinoh Pumps

Use Lansinoh® Pump to pump directly into Lansinoh® Breastmilk Storage Bags (compatible with Signature Pro®, Smartpump™ and Lansinoh Manual Breast Pump)

1) Open Breastmilk Storage Bag by tearing away tamper-evident safety seal and pulling zipper seal apart.

2) To insert expression set:

a) Tilt expression set as shown.

b) Insert one side of the expression set to align with the printed white line and flower.

c) Gently pull the other side of bag up over expression set so the clear zipper is above the expression set on all sides.

3) Gently pull the bag down until the expression set is just below the clear bottom zipper all the way around the bag. Tug gently on bottom of bag to ensure bag is secure.

4) If working with a double pump, repeat steps 1-3.

5) Pump as usual.

6) To remove bag, reverse step 2. **DO NOT PULL BAG STRAIGHT DOWN.** Securely hold the bag and gently push the zipper seal down past expression set. Maintaining a firm grip on bag, tilt and remove adapter. Seal bag.

NOTE: Do not fill past the 6oz. mark.

OR

Directions for use with Other Pumps

Use with Breastmilk Storage Adapter – visit Lansinoh.com to find out where to purchase. Compatible with most Medela®, Ameda®, Evenflo® or other standard neck pumps.

Before First Use

1. Always wash and dry hands thoroughly with soap and water before handling the adapter.
2. Sanitize by covering adapter with water and bringing the water to a boil. Allow to boil for 5 minutes.
3. Place on a clean towel to dry. Allow to dry completely before use.

1) Screw pump adapter into breast pump flange body, making sure it is tight and secure.

2) Open Breastmilk Storage bag by tearing away tamper-evident safety seal and pulling zipper seal apart.

3) To insert adapter:

a) Tilt adapter as shown.

b) Insert one side of the adapter to align with the printed white line and flower.

c) Gently pull the other side of bag up over adapter so the clear zipper is above the adapter on all sides.

4) Gently pull the bag down so the adapter is just below the clear zipper and is above printed white line. Tug gently on bottom of bag to ensure bag is secure.

5) If working with a double pump, repeat steps 1-4.

6) Pump as usual.

7) To remove bag, reverse step 3. **DO NOT PULL BAG STRAIGHT DOWN.** Securely hold the bag and gently push the zipper seal down past pump adaptor. Maintaining a firm grip on bag, tilt and remove adapter. Seal bag.

After Each Use

1. Disassemble adapter from pump.
2. Hand wash the adapter using warm soapy water. Top rack dishwasher safe.
3. Place on a clean towel to dry. Allow to dry completely before use.

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