## Congratulations On Your Choice To Breastfeed!

We are honored to support you on your breastfeeding journey. For over 30 years, Evenflo Feeding has created and continues to develop products to help you along the way. You may feel challenged at times, but

remember you are a strong and capable mommy who can achieve your breastfeeding goals! Good luck to you, mom, as you continue on your amazing adventure of motherhood! With Love, Evenflo Feeding

**Important:** Be sure to retain this manual for future reference and read it completely before using Evenflo Advanced Milk Storage Bag Adapters. Refer to your pump manual for additional instructions and warnings.


### Before First Use and After Each Use:

For best results and to avoid potential damage, hand wash (avoid boiling and high temperature dish washing). DO NOT place in the microwave. Thoroughly dry before use.

### Instructions for Use


Easy to Use – Just 4 Steps!

- 1) Screw adapter into pump flange base
- 2) Pinch adapter clips to open position
- 3) Slide open milk storage bag between clips with adapter funnel below zipper
- 4) Release adapter clips to closed position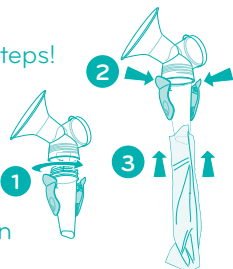

### To Release Milk Storage Bag from Adapter

- 1) Hold flange with one hand around the clip area – this same hand will be used to open clips
- 2) Press open clips and slide bag down, off of the adapter
- 3) Put flange down and seal milk storage bag according to instructions provided by your milk storage bag manufacturer

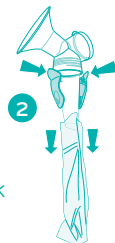**WARNING:** DO NOT use this product if it is broken or any parts are missing or become damaged. The spring clips are a potential pinch point. Contains small parts which could cause a child to choke. Keep pump parts and accessories out of children's reach. Pump and accessories are personal care items, for one user only.

**Note:** If double pumping, you can secure one flange to your body with your forearm while removing the other flange from your breast with the same hand. Follow Steps 1-3 above.

Join us online at [www.EvenfloFeeding.com](http://www.EvenfloFeeding.com)

@EvenfloFeeding

### Parent's Ally

[www.evenflofeeding.com](http://www.evenflofeeding.com)  
[www.evenflofeeding.com.mx](http://www.evenflofeeding.com.mx) (Mexico)  
USA/Canada 1-855-334-2229  
Mexico 01-800-706-1200

Evenflo Feeding, Inc.  
West Chester, OH 45069  
©2015 Evenflo Feeding, Inc.  
3020-417 23JUNE15

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