ADVANCED DOUBLE ELECTRIC REPLACEMENT PARTS KIT

Congratulations on the birth of your baby! Thank you for choosing Evenflo Feeding to support you on your breastfeeding journey. *Please refer to your pump manual for more detailed instructions and warnings.*

For more information about breastfeeding or pumping, contact a lactation expert or your physician. If you have questions about the performance or operation of your pump, please contact our Parent's Ally Resource Center.

USA/Canada: 1-855-EFI-BABY (334-2229)

www.evenflofeeding.com

Mexico: 01-800-706-12-00

www.evenflofeeding.com.mx

Important Safeguards and Warnings

- DO NOT allow small children or pets to play with these products. These parts may pose a choking or strangulation hazard.
- DO NOT use flange insert if your nipple is larger than the flange insert's center opening.
- If you are experiencing engorgement, prevent possible injury by discontinuing use of this pump and consulting a medical provider.
- Pump contains small parts which could cause a child to choke. Keep pump parts and accessories out of children's reach.
- Breast pumps are personal care items, for one user only.

Limited Warranty

- These parts are covered by a 90 day limited warranty; for further details please refer to your pump instruction manual.

Identification of Parts

Bottle

24.5-mm Flange Insert
28.0-mm Flange Insert

30.5mm Flange

Tubing Adapter

Diaphragm

Valve

Tubing

Diaphragm Cap

Sealing Disc

Bottle Cap Ring

Membrane

Before Using These Pump Parts

Before using these parts read your pump Manual. Sanitize all parts in this kit EXCEPT tubing adapter, tubing, and diaphragm cap. Refer to your pump manual for detailed instructions on how to sanitize parts, as well as use and care instructions of these parts.

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