**Advanced Double
Electric Breast Pump**

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*For video instructions about how to use and care for your pump,
visit: www.evenflofeeding.com/breastpumpinstructions or scan this QR Code.*

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Congratulations on the birth of your baby! Thank you for choosing Evenflo to support your choice to breastfeed.

For more information about breastfeeding or pumping, contact a lactation expert, your physician, or visit one of the websites listed below.

If you have questions about the performance or operation of this pump, please contact our Parent’s Ally Resource Center.

USA/Canada: 1-855-EFI-BABY (334-2229) or visit us online at www.evenflofeeding.com.

Mexico: 01-800-890-32-40 or visit us online at www.evenflofeeding.com.mx.

International Lactation Consultant Association

Lactation consultants with the IBCLC credential specialize in the clinical management of breastfeeding. There may be a fee to use lactation consultant services. To locate one near you, call: (919) 861-5577. Website: www.ilca.org

La Leche League

Provides free mother-to-mother breastfeeding support. Phone: 1-800-LALECHE (US), or (847) 519-7730. Website: www.llli.org

Office on Women's Health

Provides trained breastfeeding peer counselors for free telephone help with common issues. Phone: 1-800-994-9662. Website: www.womenshealth.gov

WIC (Women, Infant and Children Program)

Provides breastfeeding help to mothers and children enrolled in the WIC program. Website: www.fns.usda.gov

Babies in the Workplace

Website: www.babiesatwork.org

Breastfeeding and Working

Website: www.workandpump.com

Breastfeeding.com

Information on multiple topics. Website: www.breastfeeding.com

Kelly Mom Breastfeeding and Parenting

Information on multiple topics. Website: www.kellymom.com

Low Milk Supply Information and Support

For mothers experiencing low milk supply. Website: www.lowmilksupply.org

Breastfeeding After a Reduction

Information for women who are breastfeeding after a breast reduction. Website: www.bfar.org

Please Note: This pump is a personal care item and should never be shared between mothers, resold, or given from one mother to another.

Indications for Use: The Evenflo Advanced Double Electric Breast Pump is an electrically powered suction device intended to express and collect milk from a lactating woman’s breasts. This pump is intended for a single user.

Please note that this pump should not be serviced or disassembled. If you have any issues or concerns with the product, please contact our Parent’s Ally Resource Center.

Table of Contents

Important Safeguards and Warnings 2

Identification of Parts 4

Before Using Your Breast Pump 5

Assembling the Flange..... 7

Single Versus Double Pumping 8

Attaching Flange Assembly to Pump Unit 8

Power Operations 8

Using Batteries 9

Using the AC Adapter 10

Check the Fit of the Flange 10

Using the Breast Pump 11

After Every Use 14

Storing Your Breast Milk 14

Using Your Breast Milk 15

Troubleshooting 16

Limited Warranty 17

IMPORTANT SAFEGUARDS AND WARNINGS

Read all instructions before using and save for future use.

When using this pump and other electrical products, basic safety precautions should always be maintained, including the following:

Danger: To reduce the risk of electrocution

- **ALWAYS** unplug this product immediately after using.
- **DO NOT** use while bathing.
- **DO NOT** place or store this product where it can fall or be pulled into a tub or sink.
- **DO NOT** place in or drop into water or other liquid.
- **DO NOT** reach for an electrical product that has fallen into water. Unplug the product immediately.
- **NEVER** immerse pump unit or AC adapter in water or attempt to sanitize in a microwave oven.

To reduce the risk of burns, electrocution, fire or injury to persons:

- This product should never be left unattended when plugged in.
- Close supervision is necessary when this product is used near children.
- Use this product only for its intended use as described in this manual.
- **DO NOT** use attachments not recommended by the manufacturer.
- **NEVER** operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, dropped into water, or if any parts are broken or missing.
- Keep the cord away from heated surfaces.
- **DO NOT** overfill the 5-ounce collection bottle. Overfilling could damage the pump by allowing breast milk to enter the motor, AC adapter port, or battery compartment, which can result in decreased performance.
- **NEVER** use while sleepy or drowsy.
- **NEVER** drop or insert any object into any opening of the pump unit.
- **DO NOT** use outdoors with the AC adapter or operate where aerosol spray products are being used, or oxygen is being administered.
- Operate pump using **ONLY** 6 “AA” batteries (not included) or the Evenflo AC adapter provided, and plug the adapter into a standard wall outlet.
- Operate the AC adapter on 100 – 240 volts AC only. Connect only to household outlet power supplies of this description.
- **DO NOT** allow small children or pets to play with the pump unit, flange, AC adapter, etc. These parts may pose a choking or strangulation hazard.

IMPORTANT SAFEGUARDS AND WARNINGS

- Protect power cord and AC adapter cord. Route cords so that they are not walked on or pinched by items placed upon them.
- **DO NOT** mix old and new batteries. **DO NOT** mix alkaline, standard (carbon-zinc), or rechargeable (nickel-cadmium) batteries; leakage, explosion and/or fire could occur.
- **DO NOT** use the pump if the AC adapter cord or plug have been damaged.
- **DO NOT**, under any circumstances, use this AC adapter with any other equipment. Likewise, do not use the adapter from any other product on this breast pump.
- Remove batteries before storing unit for a prolonged period of time.
- **DO NOT** heat breast milk in the microwave. Microwaving can destroy nutrients in breast milk and can heat liquids unevenly resulting in burns or scalds.
- **DO NOT** re-freeze breast milk. After feeding, any unused breast milk left in the bottle can be refrigerated and used within 4 to 6 hours.
- **DO NOT** attempt to pump breast milk directly into disposable bottles with the electric breast pump. The pump is not designed to work with disposable bottles.
- **DO NOT** use this product if it is broken or any parts are missing.
- **DO NOT** use flange insert if your nipple is larger than the flange insert's center opening.
- If you are experiencing engorgement, prevent possible injury by discontinuing use of this pump and consulting a medical provider.
- Pump contains small parts which could cause a child to choke. Keep pump parts and accessories out of children's reach.
- Breast pumps are personal care items, for one user only.
- **ALWAYS** test temperature of liquid before feeding.
- **DO NOT** let child take bottle to bed or self-feed for long periods. Severe tooth decay may result from prolonged contact with liquids containing sugar such as milk, formula, or juice. Discuss proper bottle feeding with our doctor or dentist.
- If you are a mother who is infected with Hepatitis B, Hepatitis C or Human Immunodeficiency Virus (HIV), pumping breast milk will not reduce or remove the risk of transmitting the virus to your baby through your breast milk.
- To minimize contamination, keep pump parts (including bags) clean and dry when not in use.

• Save These Instructions for Future Use.


Identification of Parts


Bottle

24.5-mm Flange Insert
28.0-mm Flange Insert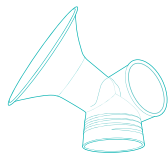

Flange

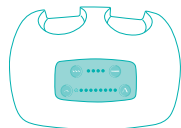Pump unit

Valve


Membrane

Sealing Disc

Bottle Cap Ring

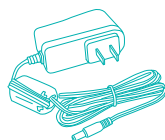AC adapter

Diaphragm

Diaphragm Cap

Tubing Adapter


Tubing

Drawstring Bag for
Wet Parts

Double Pump Bag
(Some Models Only)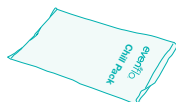


Ice Pack
(Some Models Only)


Cooler Bag
(Some Models Only)

Before Using Your Breast Pump


IMPORTANT: Before using your breast pump, disassemble and sanitize thoroughly. Always wash hands well with soap and water before handling your pump.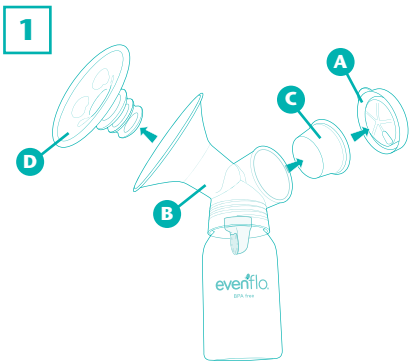

Remove diaphragm cap **A** from flange assembly **B** by pulling up on cap tab.
Remove diaphragm **C** and flange insert **D** from flange assembly.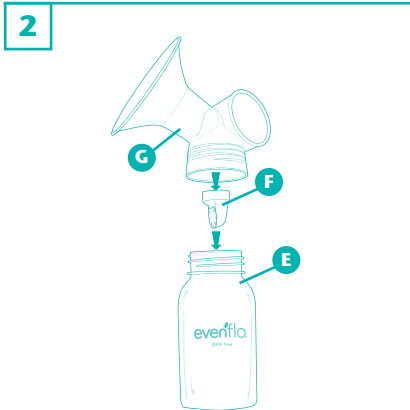

Remove bottle **E** and valve assembly **F** from the flange **G**.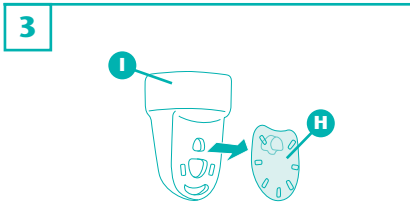

Remove membrane **H** from valve **I**.

Repeat steps 1 through 3 for second flange assembly.

Before Using Your Breast Pump


To Sanitize Parts on the Stove:

Place disassembled pump parts (except pump unit, AC adapter, tubing, tubing adapter, and diaphragm cap) in a pan of boiling water for 5 minutes. **Stir frequently. DO NOT** allow parts to rest on bottom of pan as they may melt or become damaged.

To Sanitize Parts in the Microwave:

This pump may be used with a microwave sanitation device (except pump unit, AC adapter, tubing, tubing adapter, and diaphragm cap). Please follow device manufacturer's instructions.

DO NOT BOIL OR WASH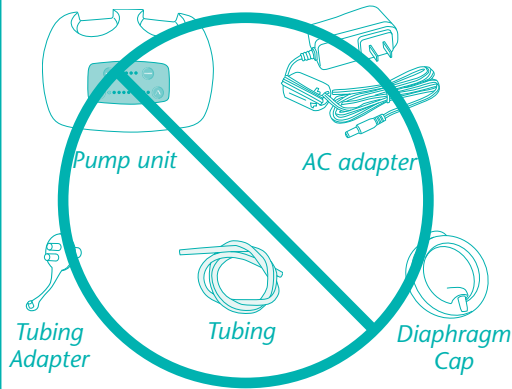

⚠ WARNING


NEVER immerse pump unit or AC adapter in water or attempt to sanitize in a microwave oven. Damage to parts and oven may occur. Do not boil or wash tubing or tubing adapter.

After Every Use

- Wipe the pump unit with a clean, damp cloth.
- Rinse the flange assembly (**all parts EXCEPT** the pump unit, tubing, tubing adapter, diaphragm cap, and AC adapter) in cool water to remove milk residue.
- Hand wash with warm, soapy water.
- Rinse thoroughly and air dry before assembly. You may also wash your flange assembly (after rinsing) on the top rack of your dishwasher.
- Use care when handling the valve/membrane assembly during cleaning to avoid damage. Before each use, visually inspect the individual components for cracks, chips, tears, discoloration or deterioration. In the event that damage to the device is observed, please discontinue use until the parts have been replaced.
- Inspect the tubing for condensation or milk after each use. If condensation is observed in the tubing, contact Evenflo for replacement parts.

Assembling the Flange


1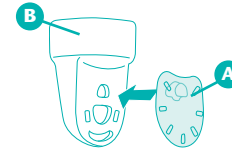

Attach the membrane **A** to the valve **B**.


Note: Make sure the membrane is attached and is lying flat against the valve, or the pump will lose suction and will not work properly. Also, make sure the membrane is not torn, as this too will cause decreased suction and the pump not to work properly.

2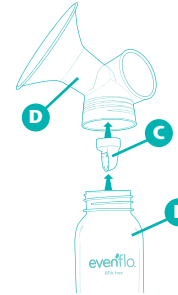

Push valve assembly **C** onto the bottom of the flange **D**. Rotate bottle **E** into bottom of flange.


3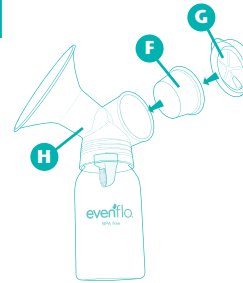

Install diaphragm **F** and diaphragm cap **G** to flange assembly **H**.

Note: Diaphragm cap can be turned to orient tubing as desired.

4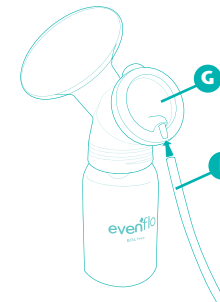


Attach tubing **I** to diaphragm cap **G**.

Note: If you intend to pump from both breasts at the same time, repeat steps 1 through 4 for the second flange.

Single Versus Double Pumping


The tubing adapter can be used to pump with one or both flanges.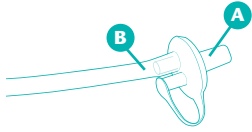

To Pump One Breast at a Time

Cap one side of the tubing adapter with the attached cap (A). Connect one tube (B) to the other side.


Note: The open side must be completely capped to ensure proper suction.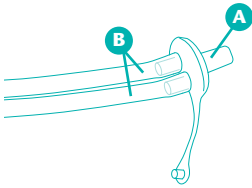

To Pump Both Breasts at the Same Time

Ensure that both tubes (B) are properly connected to the tubing adapter (A).

Attaching Flange Assembly to Pump Unit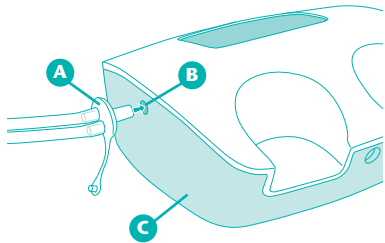

Insert tubing adapter (A) into the port (B) on the side of the pump unit (C).


Note: To minimize damage to the pump unit, always operate with a flange assembly attached.

Power Options

This pump can be powered by either 6 “AA” batteries or the supplied Evenflo AC adapter.

Note: When batteries are first installed or when AC adapter is first plugged in, the pump unit will briefly turn on then off; this is normal.

Using Batteries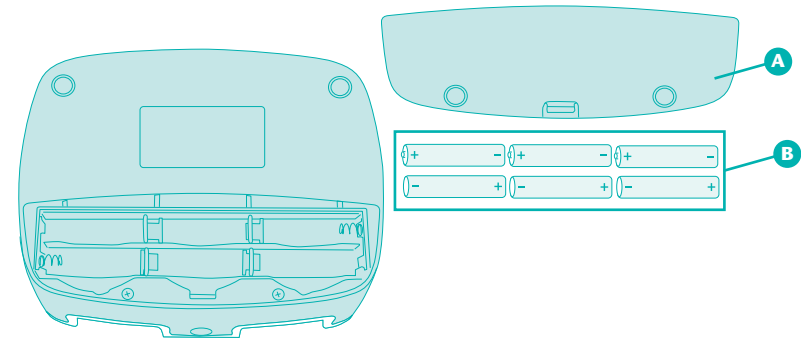


⚠ WARNING

- **DO NOT** mix old and new batteries.
- **DO NOT** mix battery types; leakage, explosion and/or fire could occur.
- Remove batteries before putting product into storage.

If you plan to use batteries, lift tab on battery door (A) and insert six alkaline “AA” batteries (B) (not included). For best performance, brand name non-rechargeable batteries are recommended. Replace battery door.

Low Battery Indication


When batteries are low, the first suction level indicator (C) will blink, but the pump will still be usable.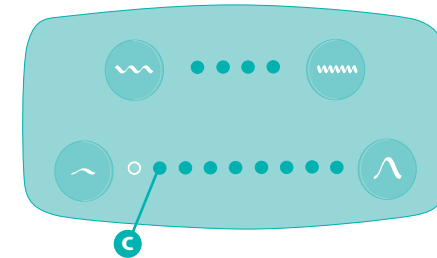

Once battery level is too low to pump, the pump will stop and not turn on until the batteries are replaced or the AC adapter is used. During this time the first suction level indicator (C) will blink, but the pump will not turn on.


⚠ WARNING

- **DO NOT**, under any circumstances, use this AC adapter with any other equipment. Likewise, **DO NOT** use the adapter from any other product on this breast pump. If you lose the AC adapter, contact Evenflo for a replacement.
- **ALWAYS** remove plug from product and wall by pulling on the plug/housing. **DO NOT** pull on the cord as this could damage the cord.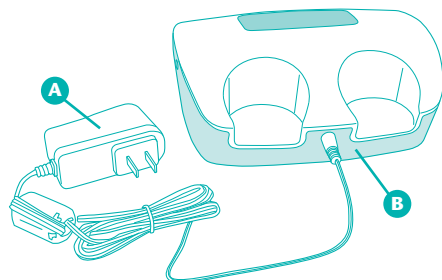

Before using the AC adapter **A**, remove batteries from pump unit. Plug AC adapter into the pump unit **B** then plug the adapter into 100 – 240V AC current only (standard household outlet). Unplug the adapter after each use.

Check the Fit of the Flange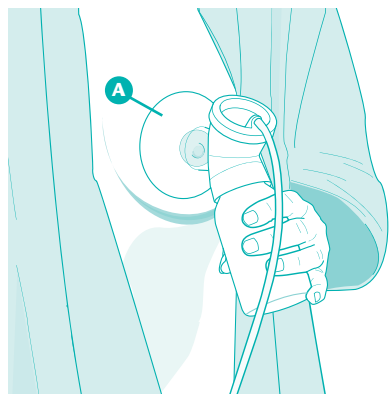

Place the flange **A** to your breast with your nipple in the exact center, as shown. Use the size that is the most comfortable and gives the best results.


Three flange sizes are included with this pump:

- Small = 24.5 mm (insert)
- Medium = 28.0 mm (insert)
- Large = 30.5 mm (no insert)


Two additional flange sizes can also be purchased separately:

- X-Large = 33.0 mm (insert)
- XX-Large = 36.0 mm (no insert)


Use the flange or appropriate flange insert that is the most comfortable and seals against your breast the best. **Note:** Use only one soft flange insert per flange.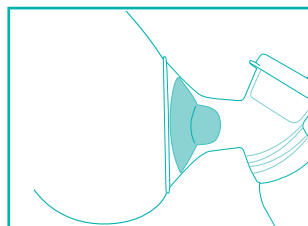

Correct flange size.
During pumping, your nipple should move freely in the breast flange tunnel. You should see space around the nipple. A minimal amount of your areola is drawn into the tunnel with the nipple.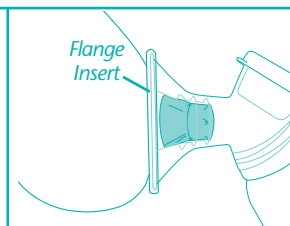

Flange is too small.
During pumping, some or all of your nipple rubs against the sides of the breast flange tunnel. Use a larger size.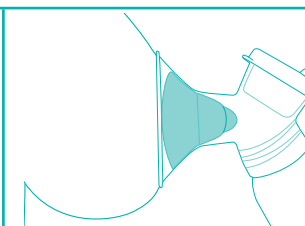


Flange is too large.
During pumping, more areola is drawn into the breast flange with your nipple. Your areola may rub against the side of the breast flange tunnel. Use a smaller size.

IMPORTANT:

- **DO NOT** use the flange insert if your nipple is larger than the center opening.
- Your breast may change over time. Should this occur, choose a different size for maximum comfort.
- **Included flange sizes may not fit all breast and nipple sizes.** If you are having issues after trying all included sizes, please contact our Parent's Ally Resource Center for assistance. Larger flanges can be purchased separately.

Using the Breast Pump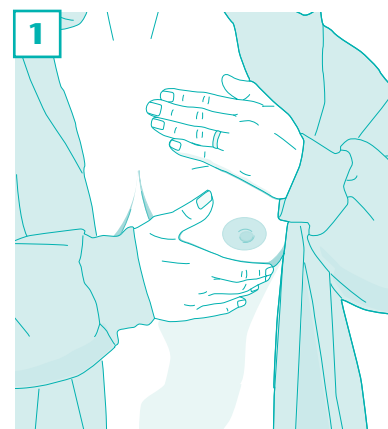


To maximize your pumping results, stimulate your let-down reflex by:

- Relaxing and thinking about your baby;
- Gently massaging your breasts, starting nearest your ribs and working down toward the nipple;
- Applying warm, moist compresses.

Continue until you can hand-express a few drops of milk.


Now you should be ready to use your pump.

2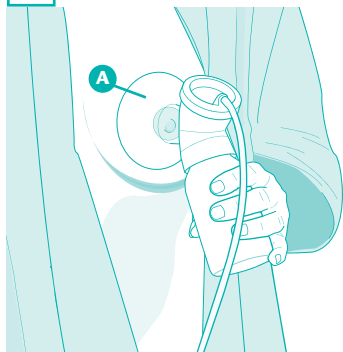

To ensure a good seal, place the flange **A** gently, but firmly, against your breast, with your nipple directly in the center, as shown. Hold the bottle upright to ensure that milk flows into the bottle.

3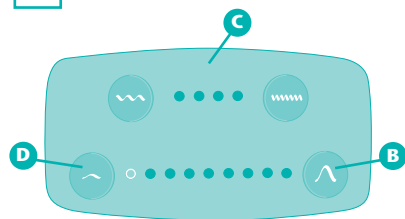

Turn On the Pump

Press the suction increase button **B** on the pump unit **C**. The pump will start at the lowest suction setting and highest speed.

To Increase suction

Press the suction increase button **B** to reach the level that is most comfortable for you.


To Decrease suction

Press the suction decrease button **D** to decrease suction.

IMPORTANT: If you feel pain in your breast or nipple, turn the pump off (see next page) or slide a finger between the flange and your breast to immediately break the suction. If you feel discomfort at every suction level, turn the pump off and contact your healthcare provider for assistance.

Note: Pump will automatically shut off after 40 minutes.


4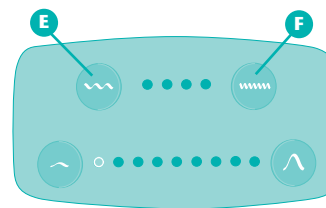

When milk begins flowing (let-down begins), press the speed decrease button **E** to help the milk flow faster. As milk flow slows, stimulate another let-down by increasing speed using the speed increase button **F**. Continue pumping until milk is no longer flowing or bottle is full.

Note: Do not overfill the 5-ounce collection bottle; when freezing your milk for storage, space is needed in the bottle for the milk to expand.

5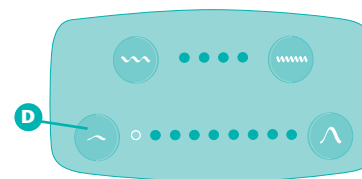


Turn off the pump

After breast milk has been expressed, hold suction decrease button **D** until the suction level reaches the lowest setting. Release button, then press and hold suction decrease button **D** again until pump turns off (approximately 1 to 2 seconds).

If necessary, slide a finger between the flange and your breast to break any remaining suction, then remove the flange.

6

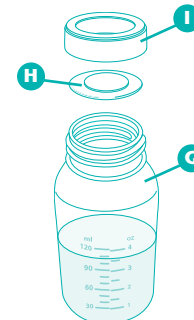Carefully remove bottle **G** from flange. Place the sealing disc **H** into the cap ring **I** and screw onto the bottle securely. Your breast milk is now ready for storage (see page 14 for information on storing breast milk).

Note: When using as a single pump on one breast at a time, repeat steps 1 through 6 on your other breast.

⚠ WARNING

NEVER immerse pump unit or AC adapter in water or attempt to sanitize in a microwave oven. Damage to parts and oven may occur. Do not boil or wash tubing or tubing adapter.

- To ensure that your breast pump performs well, follow these steps after every use:
- Wipe the pump unit with a clean, damp cloth.
 - Rinse the flange assembly (**all parts EXCEPT** the pump unit, tubing, tubing adapter, diaphragm cap, and AC adapter) in cool water to remove milk residue.
 - Hand wash with warm, soapy water.
 - Rinse thoroughly and air dry before assembly. You may also wash your flange assembly (after rinsing) on the top rack of your dishwasher.
 - Use care when handling the valve/membrane assembly during cleaning to avoid damage. Before each use, visually inspect the individual components for cracks, chips, tears, discoloration or deterioration. In the event that damage to the device is observed, please discontinue use until the parts have been replaced.
 - Inspect the tubing for condensation or milk after each use. If condensation is observed in the tubing, contact Evenflo for replacement parts.

Storing Your Breast Milk

1. After sealing your bottle tightly with the sealing disc and cap ring, label the bottle with the date and time the milk was expressed.
2. Use the chart below as a guide for storing your breast milk for future use.
Note: In hot weather, chill milk immediately to prevent spoilage.

Breast Milk Storage Guidelines*

| | Room Temperature | Refrigerator | Self-contained Refrigerator Freezer | Deep Freezer |
|---------------------------------------|----------------------------------|-----------------------------------|-------------------------------------|-----------------------------|
| Freshly Expressed Breastmilk | 4 hours at 73 – 77°F (23 – 25°C) | 5 – 7 days at 32 – 39°F (0 – 4°C) | 3 – 4 mos. at 0°F (-19°C) | 6 – 12 mos. at 0°F (-19°C) |
| Thawed Breastmilk (previously frozen) | DO NOT store | 24 hours | NEVER re-freeze thawed milk | NEVER re-freeze thawed milk |

Note: Breast milk should be stored in the main refrigerator or freezer compartment, away from the door, to avoid changes in temperature.

* Human Milk Banking Association of North America. Best practice for expressing, storing and handling human milk in hospitals, homes and child care settings. Raleigh, NC: 3rd Edition, 2011

For models that include a cooler bag:

1. Place frozen ice packs in the included cooler bag.
2. Place the sealed bottle(s) inside and zip the bag closed.
3. When frozen ice packs are used and the cooler bag is kept at room temperature of 68 – 70° F (20 – 21° C), breast milk can be safely stored for up to 10 hours. **Note:** Avoid storing the cooler bag in a hot closet, cabinet, or car.

Using Your Breast Milk

1. Thaw slowly in the refrigerator. Avoid letting milk sit out at room temperature to thaw. (For quicker thawing, hold container under running water – start with cool water and gradually increase the temperature.)
2. Just before feeding, warm the milk by:
 - Using a bottle warmer
 - Holding the sealed bottle under warm, running water, or
 - Heating water in a cup or other small container, then placing the sealed bottle in the water to warm.
3. The milk fat may separate during thawing. If so, gently swirl milk (**do not shake**) to mix before offering it to baby.
4. Push an Evenflo® nipple through the cap ring and screw onto the bottle.
5. Always test the temperature of the milk before feeding.

IMPORTANT!

- **NEVER microwave human milk or heat directly on the stove.**
- Previously frozen milk may be kept in the refrigerator for up to 24 hours after it has finished thawing. **DO NOT re-freeze.**
- **If baby does not finish milk at one feeding,** it may be refrigerated and offered at the next feeding before it is discarded.
- **DO NOT** pour freshly pumped breast milk onto frozen breast milk because thawing can occur.
- Fill containers 3/4 full, as milk expands with freezing.

Problem: Pump won't turn on.

Possible Cause: No batteries or low batteries.

Action: Make sure fresh batteries are installed correctly (p. 9). If suction level 1 light is flashing when the pump is off, the batteries are too low to effectively pump.

Possible Cause: AC adapter not connected properly.

Action: Check adapter connections (p. 10).

Problem: Little or no suction

Possible Cause: Suction setting too low.

Action: Choose a higher suction level by pressing the suction increase button (p. 12).

Possible Cause: Valve not assembled to flange.

Action: Mount the valve firmly to flange (p. 7).

Possible Cause: Membrane worn, torn or not assembled to valve.

Action: Replace membrane or ensure that it is connected properly to the valve (p. 7). To maintain optimal performance with daily use, it is recommended that membranes be replaced every 3 months.

Possible Cause: Dried milk in pump parts.

Action: Follow cleaning instructions. This pump can be easily disassembled and washed (pp. 6 and 14).

Possible Cause: Membrane not sealing.

Action: Make sure valve and membrane are clean, with no debris on the surface of the seal. Make sure membrane lies flat against the surface of the valve (p. 7).

Possible Cause: Tubing or tubing adapter is loose or not connected.

Action: Check assembly of flange. All parts must be assembled properly and fit together snugly, without gaps, for the pump to provide maximum comfortable suction (pp. 7 and 8).

Possible Cause: Not a good seal on your breast.

Action: Reposition the flange to get a tighter seal. If the breast becomes too wet with milk, you may need to dry it off and start again (p. 12).

Problem: Milk won't flow into the bottle.

Possible Cause: The valve is not opening.

Action: Clean the valve thoroughly and be sure it opens (p. 7).

Still having problems? Call our Parent's Ally Resource Center for help.

If you have any questions, comments, or concerns about our Evenflo Feeding, Inc. ("Evenflo") breast pumps (the "Products"), please contact our Parent's Ally Resource Center at 1-855-EFI-BABY (334-2229) or **www.evenflofeeding.com**.

This is a personal care item for one user only. For hygienic reasons, it **CANNOT BE RETURNED TO THE STORE ONCE PACKAGE HAS BEEN OPENED**. Use of this Product by more than one person will also void the following express limited warranty.

From the original purchase date of this Product, for a period of one year for the Product's motor and 90 days for all other Product components and accessories, Evenflo provides a warranty to the original end user ("Purchaser") against defects in material or workmanship. Evenflo's sole obligation under this express limited warranty shall be, at Evenflo's option, to repair or replace any Product, component or accessory that is determined to be defective by Evenflo and determined to be covered by this warranty.

Repair or replacement as provided under this warranty is the exclusive remedy of the Purchaser. **Proof of purchase in the form of a receipted invoice or bill of sale evidencing that the Product is within the warranty period must be presented to obtain warranty service.** This express limited warranty is extended by Evenflo ONLY to the original Purchaser of the Product and is not assignable or transferable to subsequent purchasers or end users of the Product. For warranty service, contact Evenflo's Parent's Ally Resource Center at 1-855-EFI-BABY (334-2229) or **www.evenflofeeding.com**.

EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE ON THIS PRODUCT IS HEREBY DISCLAIMED. EVENFLO SHALL NOT BE LIABLE FOR ANY INCIDENTAL, CONSEQUENTIAL, SPECIAL OR PUNITIVE DAMAGES OF ANY KIND FOR BREACH OF THE EXPRESS LIMITED WARRANTY ON THIS PRODUCT, OR ANY WARRANTY IMPLIED BY OPERATION OF LAW. OTHER THAN THE EXPRESS LIMITED WARRANTY SET FORTH ABOVE, THERE ARE NO OTHER WARRANTIES THAT ACCOMPANY THIS PRODUCT AND ANY ORAL, WRITTEN OR ANY OTHER EXPRESS REPRESENTATION OF ANY KIND IS HEREBY DISCLAIMED.

Parent's Ally

• USA/Canada: 1-855-EFI-BABY (334-2229); **www.evenflofeeding.com**

• Mexico: 01-800-890-32-40; **www.evenflo.com.mx**

Manufactured and Distributed by Evenflo Feeding, Inc.

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Evenflo México S.A. de C.V.